




**NOVAKOR**  
GLOBAL

[www.novakorglobal.com](http://www.novakorglobal.com)  
[info@novakorglobal.com](mailto:info@novakorglobal.com)



**High and Low Molecular weight HA:  
The Perfect Duo for Skin Perfection**

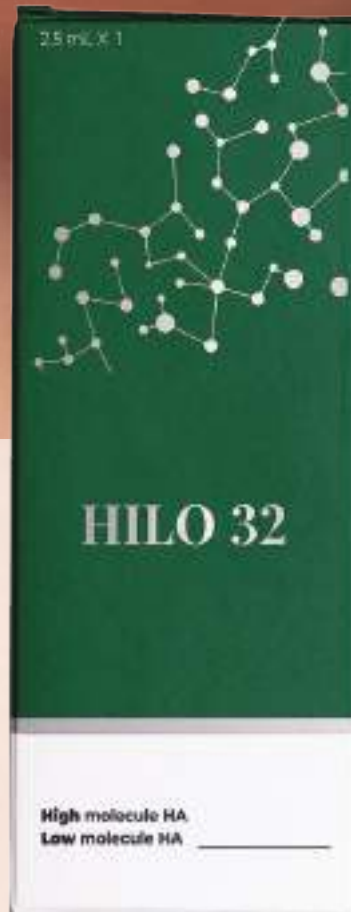

## What is H-HA & L-HA ?

This non cross-linked hyaluronic acid skin booster is developed with cutting - edge technology to improve wrinkles.


Utilizing a proprietary high - concentration manufacturing method, it overcomes the limitations of traditional low-elasticity (monophasic) and low-viscosity (biphasic) HA formulations.

This makes it highly effective for both wrinkle treatment and facial contour enhancement




## MOLECULAR WEIGHT HYALURONIC ACID

Long-lasting effect,  
Enhancement of collagen synthesis

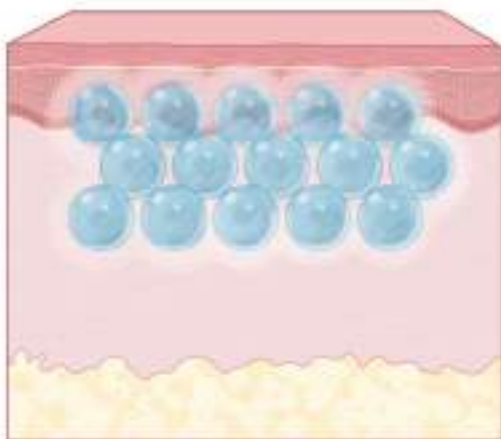

HIGH MOLECULAR (H-HA)

Instant hydration

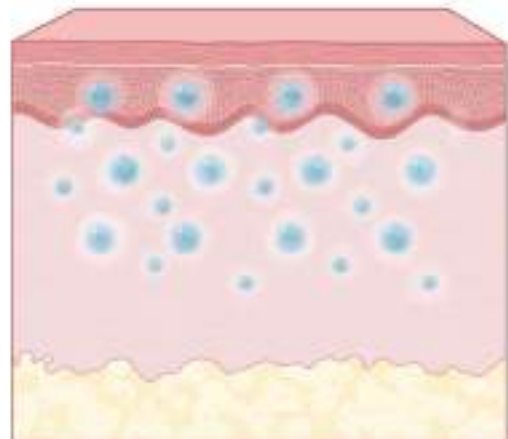

LOW MOLECULAR (L-HA)

**H-HA:** The High molecular weight hyaluronic acid increases skin volume, delivering a noticeable lifting effect

**L-HA:** The Low molecular weight hyaluronic acid penetrates deeply, delivering intensive hydration to the epidermis.

# WHY Hilo 32 / Hilo 32 Plus?



Lifting



Returns volume



Restoring Hydration



Tissue Remodeling



Stimulate collagen synthesis




Elasticity Improvement



Balance Recovery



Skin Regeneration

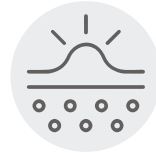

Anti Inflammatory



Skin Tightening



# Hilo 32

Non cross-linked H-HA & L-HA

**Package:** 2.5 ml per syringe

**Hilo 32 excels in providing top-tier hydration and lifting effects, making it an excellent choice for individuals seeking immediate plumping and firming of the skin.**

Its high effectiveness in improving skin elasticity also contributes to a more youthful appearance.

However, it offers moderate benefits in anti-inflammatory properties and skin regeneration, making it suitable for those primarily focused on hydration and lifting.

**Recommended number of treatments:** 2-3

**Interval between treatments:** 1 month

**Shelf Life:** 36 months

**Storage:** Store at room temperature (1-30°C), avoid direct sunlight



## Composition:

40 mg of High Molecular HA

40 mg of Low Molecular HA




## Injection Depth:

Epidermis, Dermis

## Effectiveness:

Hydration	Lifting	Volume	Elasticity	Balance Recovery	Skin Regeneration
★★★★★	★★★★★	★★★★★	★★★★☆	★★★★★	★★★☆☆

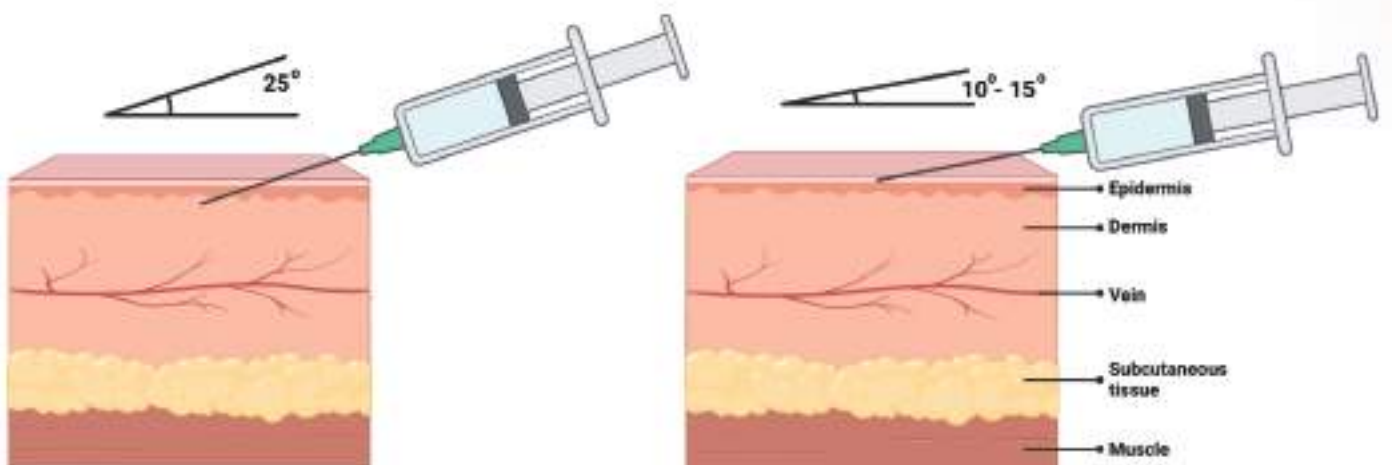



# Hilo 32 Plus

Non cross-linked H-HA & L-HA

**Package:** 2.5 ml per syringe

**Hilo 32 Plus is designed for comprehensive mesotherapy, providing excellent benefits across multiple areas.**

While its hydration and lifting effects are strong, its true strengths lie in enhancing skin elasticity, reducing inflammation, and promoting skin regeneration.

This makes it ideal for users looking for a well-rounded product that addresses various skin concerns, including aging and sensitivity.

**Recommended number of treatments:** 2-3

**Interval between treatments:** 1 month

**Shelf Life:** 36 months

**Storage:** Store at room temperature (1-30°C), avoid direct sunlight



## Composition:

40 mg of High Molecular HA

40 mg of Low Molecular HA

40 mg Succinic acid

Peptide


## Injection Depth:

Epidermis, Dermis



## Effectiveness:

Hydration	Lifting	Volume	Elasticity	Balance Recovery	Skin Regeneration
★★★★☆	★★★★☆	★★★★☆	★★★★★	★★★★★	★★★★★

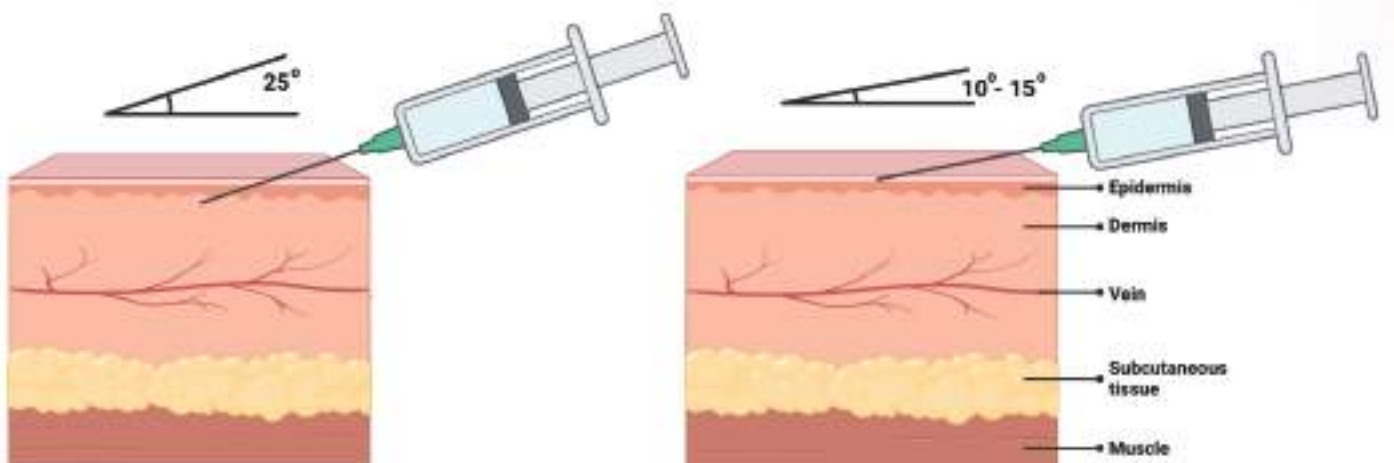


# Facial remodeling 6 points BAP technique




With just 6 BAP injection points per side of the face, Hilo32/Hilo32 Plus diffuses through multiple skin layers, covering broad areas of the face, and delivering a natural and long-lasting refreshed look.

**Recommended Needle size:** 27G/30G  
**Product quantity per Point:** 0.2 ml  
**Product quantity per Treatment:** 2.4 ml  
**Injection depth:** 3 mm  
**Angle of Inclination:** 0-45°  
**Interval between treatment:** 4 weeks.  
**Frequency:** 2-3 times per year  
Avoid blood vessel locations

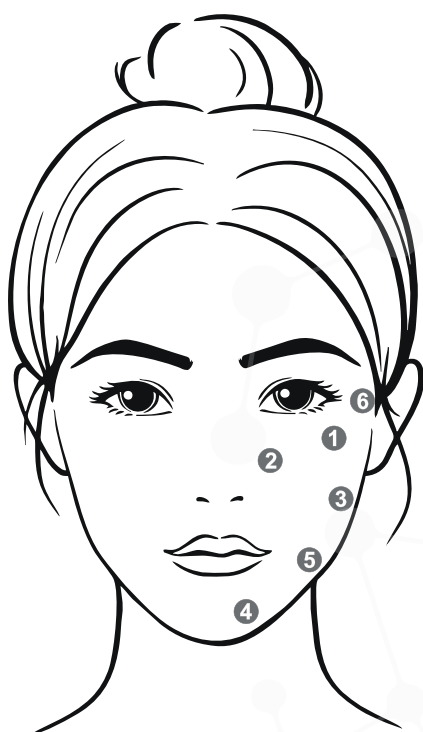

# Facial BAP Technique measurements

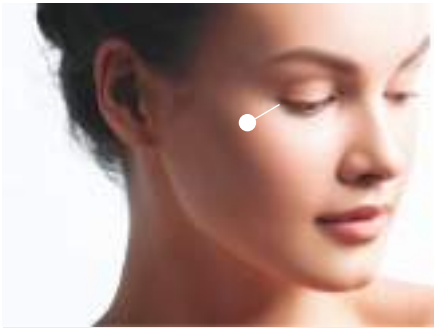

## 1. Zygomatic Protrusion

Identify the zygomatic protrusion at least 2 cm away from the external corner of the eye


## 2. Nasal Base

Draw a line connecting the nostril and tragus  
Draw a perpendicular line starting from the pupil  
Locate the injection point at the intersection of the 2 lines




## 3. Tragus

Locate the tragus and mark a point 1 cm anterior to the bottom of the tragus.- This point serves as a reference for specific treatment related to the tragus area

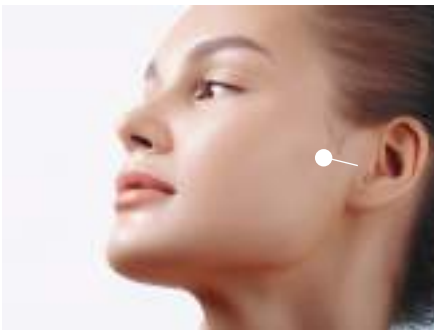


## 4. Chin

Draw a vertical line in the center of the chin-Draw a perpendicular line one third from the top of the vertical line-From the point of intersection move 1.5 cm towards the oral commissures  
This precise marking assists in determining specific treatment areas on the chin

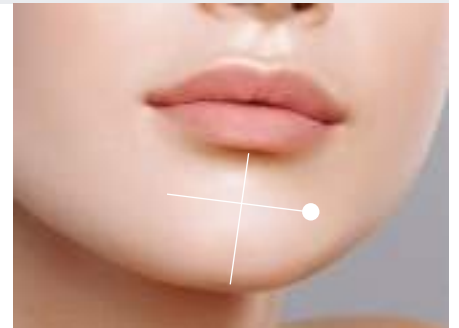


## 5. Mandibular Angle

Locate the injection point 1 cm above the mandibular angle

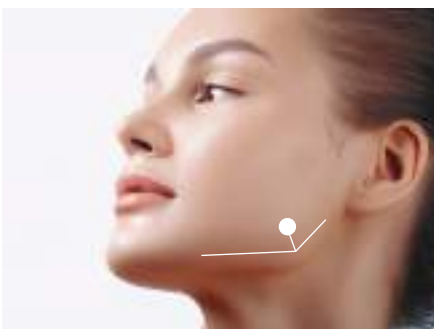

## 6. Temporalis

Locate the injection point 4.5 cm from the zygomatic arch

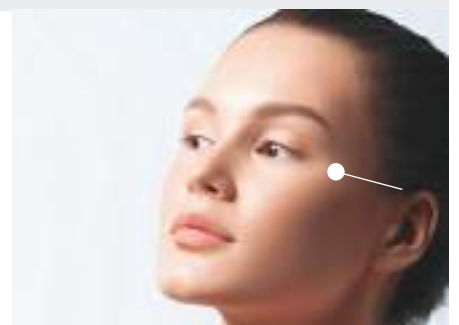

# Neck remodeling 12 points BAP technique

The 12 BAP injection points for the neck are carefully designed to avoid injury to vital structures, delivering anti-aging benefits from the inside out.



**Recommended Needle size:** 27G/30G

**Product quantity per Point:** 0.2 ml

**Product quantity per Treatment:** 2.4 ml


**Distance 1.5 cm from the center of the clavicle**

**Injection depth:** 3 mm

**Angle of Inclination:** 0-45°

**Interval between treatment:** 4 weeks.


**Frequency:** 2-3 times per year Avoid blood vessel locations

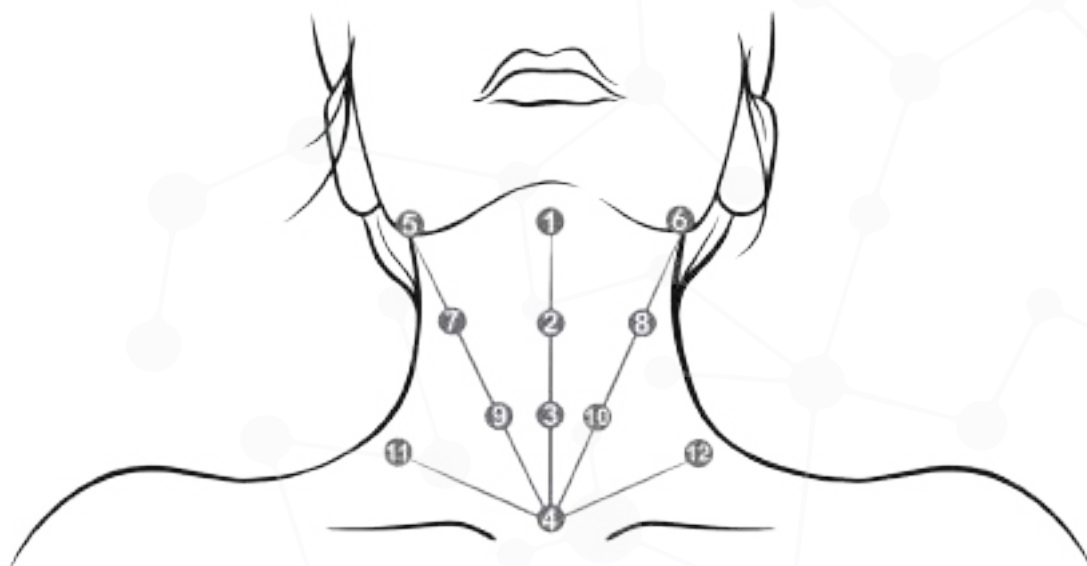




# Hand remodeling: Fanning 10 points BAP technique

---

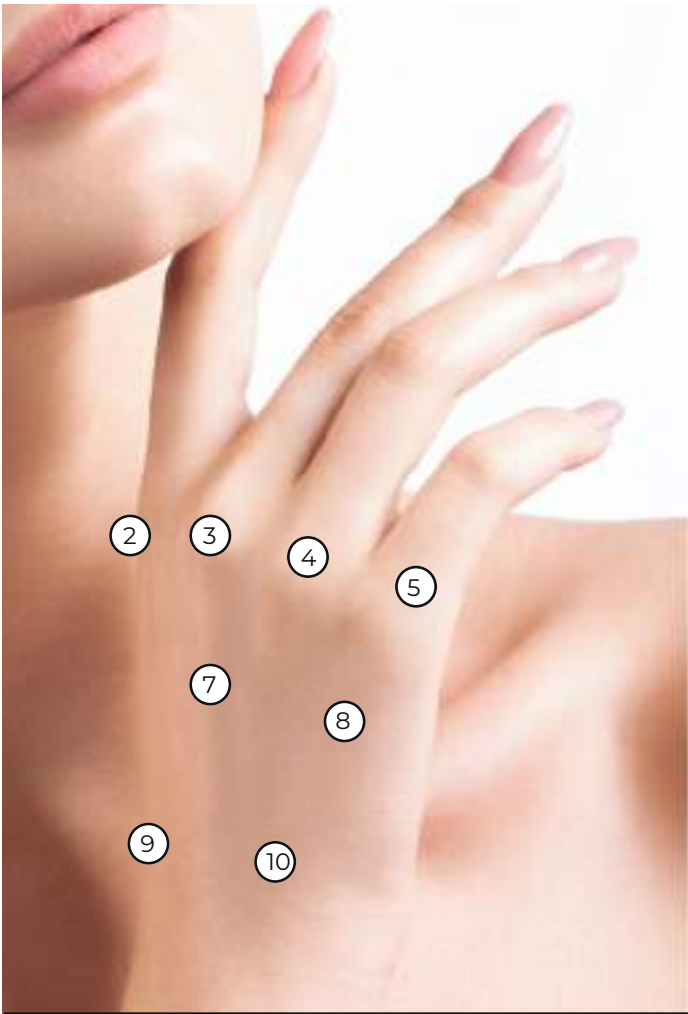


The 10 BAP injection points for the hands are meticulously placed to ensure safety and effectiveness, delivering comprehensive anti-aging benefits and rejuvenation from within.

- Recommended Needle size:** 27G/30G
- Product quantity per Point:** 0.2 ml
- Product quantity per Treatment:** 4 ml (both hands)
- Distance 1.5 cm from the middle of the middle finger
- Injection depth:** 3 mm
- Angle of Inclination:** 0-45°
- Interval between treatment:** 4 weeks.
- Frequency:** 2-3 times per year
- Avoid blood vessel locations**

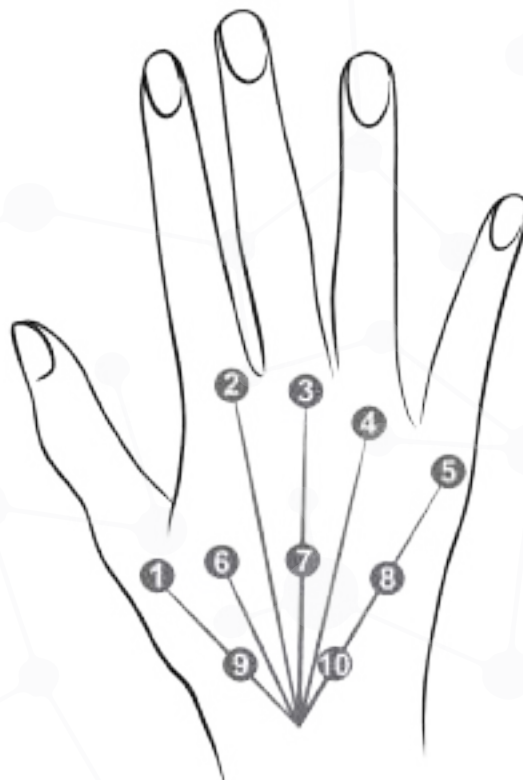

# Inner arm 7 points BAP technique

With 7 carefully selected BAP injection points, the inner arm receives a concentrated treatment that promotes skin rejuvenation and smoothness, effectively addressing signs of aging in this delicate area.

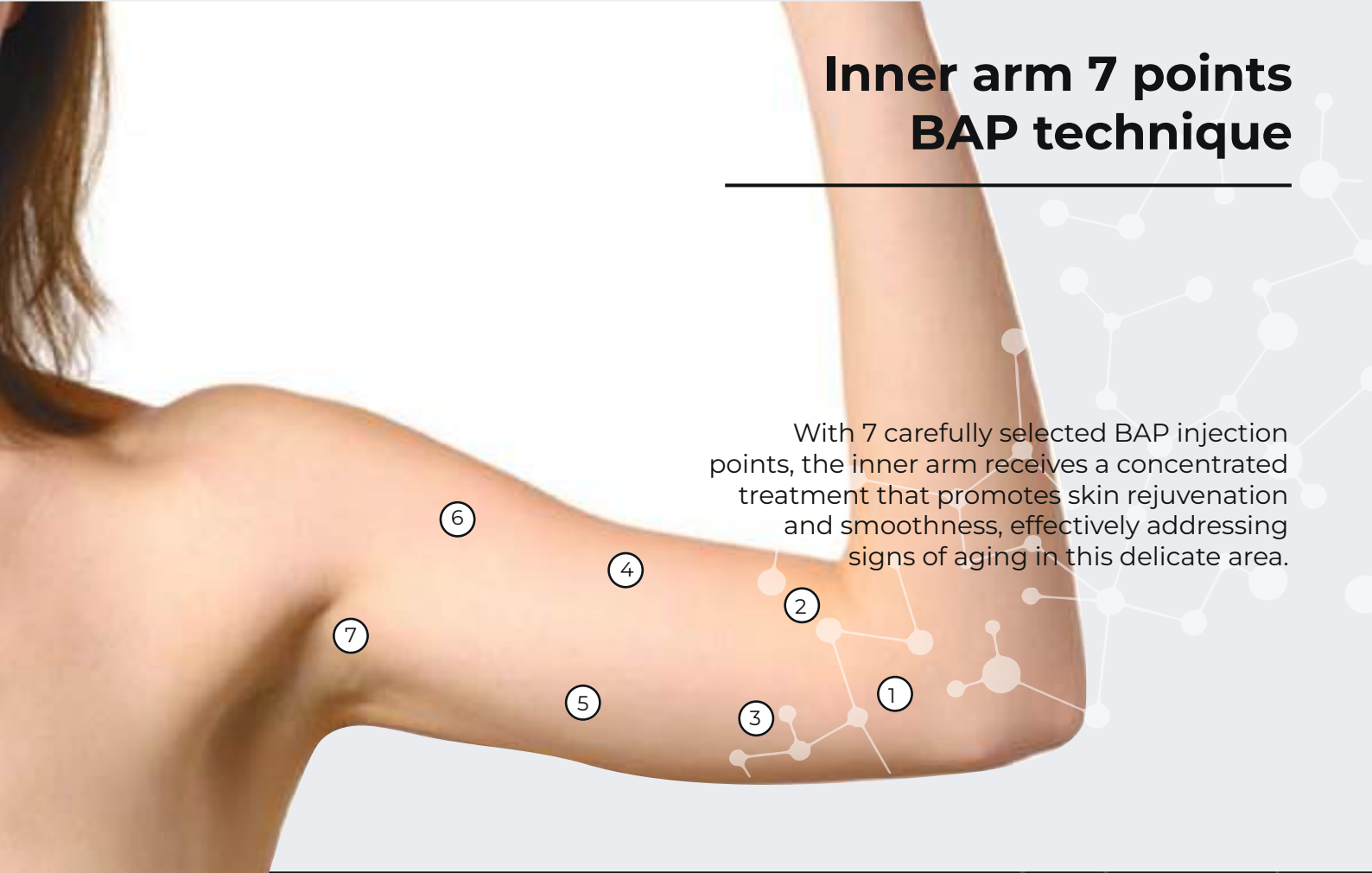

- Recommended Needle size:** 27G/30G
- Product quantity per Point:** 0.2 ml
- Product quantity per Treatment:** 3 ml (both arms)
- Injection depth:** 3 mm
- Angle of Inclination:** 0-45°
- Interval between treatment:** 4 weeks.
- Frequency:** 2-3 times per year
- Avoid blood vessel locations**






## H-HA + L-HA

The Perfect Duo for Skin Perfection



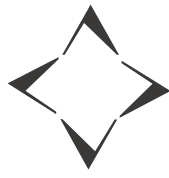

NOVAKOR  
GLOBAL

# HILO 32 & HILO 32 plus

High and Low Molecular weight HA:  
The Perfect Duo for Skin Perfection

 [www.novakorglobal.com](http://www.novakorglobal.com)

 [info@novakorglobal.com](mailto:info@novakorglobal.com)

MADE IN KOREA